

## CAUTION

Patients should be instructed to inspect their hand, wrist, and product before and after each use.

## WASHING INSTRUCTIONS

1. This product may be cleaned using a mild detergent and cold water. Aluminum stay should be removed for cleaning. Do not use bleach or strong detergents. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater

# N01a, N01b

## Medex Soft Collar

User Guide &  
Application Instruction



EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd  
Suite B, 29 Harley Street,  
LONDON, W1G 9QR, U.K.  
[www.CE-Marking.EU](http://www.CE-Marking.EU)



Council Directive  
93/42/EEC of  
14 June 1993  
concerning Medical Device

MANUFACTURED BY

MEDEX ORTHOPAEDIC & MEDICAL SUPPLIES.

T: +852-2656 8211 A: Unit D, 13/F, Block 2 Tai Ping Industrial Centre, Tai Po, HongKong

F: +852-2664 0885 W: [www.Medex-intl.com](http://www.Medex-intl.com)

# Medex

### N01a Soft Collar - Universal type

### N01b Soft Collar - Serpentine type

1. Detach velcro closure
2. Center the V-notch of the collar underneath the chin.
3. Faster velcro closure and secure to a snug fit.

