

FITTING INSTRUCTIONS

THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product, it is fundamental that the correct size adequate for each patient or user is chosen (enclosed in the packet is a diagram with all the sizes and the equivalent in centimetres). An excess compression can produce intolerance, therefore we recommend that the compression be regulated to an optimum degree.

To fit observe the following aspects:

Measure the height of the neck in the upright position and find the right size according to the chart in the box. Place the collar around the neck making sure that it is positioned correctly under the chin for optimum comfort. Tighten with the Velcro strap to as far as is comfortable.

PRECAUTIONS

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

Medex

LAUNDERING AND CARE

1. Remove any panels and/or metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Do not use any substance containing Chlorine

WARNING

1. These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.
2. The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.
3. For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU

MANUFACTURED BY

MEDEX INTL (HOLDINGS) LIMITED

T: +852-2665 3666 A: P.O. Box 603 Tai Po Post Office Hong Kong
F: +852-2665 6660 W: www.Medex-intl.com E: info@medex-intl.com



Council Directive
93/42/EEC of 14 June 1993
concerning Medical Device

Medex-intl.com